

For  
Pharmacists  
& Techs



# TOPI-CLICK®

## Filling Instructions\*

Read all information, specifications and instructions carefully before dispensing in the Topi-Click™

Step 1



Fill the Topi-Click® container with the desired amount of topical preparation using a spatula, EMP jar, or filling system.

Step 1a



If needed while filling, stop every third of the fill to tap the Topi-Click® on the palm of your hand or counter to help settle the topical.

Step 2



Put the applicator pad (cap with hole) on the top of the Topi-Click® by aligning the inside key tabs with the slots on the top of the container and snap on tight.

Step 3



# **CLICKS**  
Prime the Topi-Click® by turning the bottom until topical is dispensed and wipe off topical.

### Step 3a: VERY IMPORTANT TO LEAVE IN HOME POSITION



Priming: When rotating base, watch the Visual Indicators

Indicator Node on Body is fixed at the Click Point



Indicator Column moves with Base. This shows Node in line with column.

**LEAVE IN THIS POSITION**  
**AFTER EVERY CLICK**  
(The Home Position)

Step 4



Snap on the outer cap. (if the cap doesn't fit tightly, it's an indication that the applicator pad in #2 is not snapped down all the way).

Step 5



Apply center of label to the side of the Topi-Click® container which is opposite the "REFILL" and graduation marks.

For more information on dispensing amounts per click see the [Specifications](#) document or go to [www.Topi-Click.com](http://www.Topi-Click.com). See the [Dispensing Instructions](#) sheet on how to apply a dose from the Topi-Click®.

<sup>2</sup>Note: The graduation marks are for reference only & can be used with the bottom of the plunger (Fig. 2). Some bases will need additional calibration and some are not practical to use with the Topi-Click®.

\*As required for compounded prescription pharmaceuticals; verify the formulas and the weight to volume

Downloadable at [www.Topi-Click.com](http://www.Topi-Click.com)

The Topi-Click® product may be improved over time based on customer feedback on usage. Custom Rx TDA, LLC and/or the patent holders are not liable for any damages arising from the products dispensed in the Topi-Click® or its improper use.



Protected by U.S. Patent No. 7,213,994.

TOPI-CLICK

1309 Crooked Branch Trail  
Woodstock, GA 30189-6830  
[www.Topi-Click.com](http://www.Topi-Click.com)  
770-827-0472



All items © 2006-2011 by Custom Rx TDA, LLC.  
All rights reserved Ver: 72010