

For patients!



TOPI-CLICK®

DISPENSING INSTRUCTIONS*

Read all of this information and instructions carefully before dispensing from the Topi-Click™.

Step 1



Pop off the cap of the Topi-Click® container by placing your thumb under the tab on the protective outer cap and pressing upward.

Step 2



Turn the base to dispense the number of clicks specified on label.



When rotating base, watch the Visual Indicators

Indicator Node on Body is fixed at the Click Point



Indicator Column moves with Base. This shows Node in line with column.

LEAVE IN THIS POSITION
AFTER EVERY CLICK
(The Home Position)

Step 3



Apply treatment by rubbing the cream, which is now on the top of the applicator pad, to the area of skin as directed.

Step 4



Prevent evaporation of next dose, by snapping the cap on completely.

Step 5



The **bottom** of the plunger is the indicating line; when it reaches the embossed line **above** the word "Refill," there are approximately 12 clicks left. Approx. 32 clicks remain at the line **below** the word "Refill" as shown above.

The Topi-Click® is a one-time use container and not refillable.

Download at www.Topi-Click.com

The Topi-Click® product may be improved over time based on customer feedback on usage. Custom Rx TDA, LLC and/or the patent holders are not liable for any damages arising from the products dispensed in the Topi-Click® or its improper use.



Protected by U.S. Patent No. 7,213,994.

TOPI-CLICK

1309 Crooked Branch Trail
Woodstock, GA 30189-6830
www.Topi-Click.com
770-827-0472



All items © 2006-2011 by Custom Rx TDA, LLC.
All rights reserved Ver: 72010